VERSION FRANCAISE ..... 2

ENGLISH VERSION ..... 8

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**VERSION FRANCAISE**

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ENGLISH VERSION

**ADECHOTECH**  
Bringing Ultrasound Everywhere

**MELODY PATIENT – MELODY EXPERT - MEV**

**SAFETY AND PERFORMANCE INFORMATION**

English

This safety and performance information applies to all devices in the MELODY range:

- MELODY PATIENT
- MELODY EXPERT
- MELODY EXPERT VIRTUAL or (MEV)

## Performance information

### Clinical Benefit

MELODY contributes to have a positive impact on patient management or public health by optimizing and facilitating access to quality ultrasound for patients living in isolated medical areas (where the presence of an imaging expert cannot be ensured).

The several clinical studies conducted with the MELODY Solution have not detected any undesirable side effect due to its use.

### Clinical performance

MELODY is intended to hold the external ultrasound probe of an ultrasound equipment and provide a remotely-located expert sonographer, the ability to manipulate the probe in real time through a communication network to obtain the ultrasound views required to assess the patient.

The MELODY robot frame at the remotely-located patient is prepared and positioned by a trained assistant at the direction of the expert sonographer through the third-party videoconferencing system. Sonograms are transmitted from the patient location from cleared ultrasound equipment through digital telecommunication to the expert sonographer.

## Safety Information

### Contraindications

The MELODY system is not indicated for handling endocavitary probes.

If the patient's skin has any lesion (wound, burn...), the MELODY Solution must not be used on these injured parts.

### Electrical Safety

The MELODY system complies with applicable electrical safety standards: IEC 60601-1 (Ed 3.2).

**WARNING:** To prevent any risk of electric shock, this device must only be connected to a mains network fitted with protective grounding. There is a risk of electrocution if the mains supply adaptor is not correctly grounded. Grounding reliability can only be ensured if the system is connected to a socket complying with hospital standards. Do not remove or uncoil the ground wire. Moreover, the system must be used with the power cable supplied.

### Electromagnetic Safety

The MELODY system complies with applicable electromagnetic safety standards: IEC 60601-1-2 (Ed 4.1).

**WARNING:** Use of the MELODY system adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, the MELODY system and the other equipment should be observed to verify that they are operating normally.

**WARNING:** Use of accessories, transducers and cables other than those specified or provided by AdEchoTech could result in increased electromagnetic emissions or decreased electromagnetic immunity of the MELODY system and result in improper operation.

**WARNING:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the MELODY system including cables specified by AdEchoTech. Otherwise, degradation of the performance of this equipment could result.

### Network/Data Coupling

**WARNING:** Connection of the system to a network/data coupling that includes other equipment could result in previously unidentified risks to patients, operators or third parties. The responsible organization should identify, analyse, evaluate and control these risks.

**WARNING:** Subsequent changes to the network/data coupling could introduce new risks and require additional analysis. Changes to the network/data coupling include: changes in network configuration, connection of additional items, disconnection of items, update of equipment and upgrade of equipment.

### Internet Failure

To detect failures related to the Internet network, AdEchoTech has implemented indicator lights to display the status of the network in real time. It is therefore very important to ensure that the minimum network characteristics are met.

In situations where a diagnosis is necessary for therapeutic treatment within a very short time, it is up to the doctor to analyse the benefit / risk ratio for his patient, taking into account the information mentioned above.

## Use Environment

**WARNING:** MELODY system is not suitable to be used in an oxygen rich environment.

**WARNING:** MELODY system is not suitable to be used with flammable anaesthetics products.

**WARNING:** MELODY system is not suitable to be used in conjunction with flammable agents.

### *MELODY Patient*

The MELODY Patient system is suitable to be used in professional healthcare facility environments: eg. physician offices, clinics and hospitals except where the intensity of EM disturbances is high.

The MELODY Patient system is not suitable to be used in home healthcare environment and emergency medical services.

### *MELODY Expert et MEV*

The MELODY Expert and MEV system are suitable to be used in professional healthcare facility environments: eg. physician offices, clinics and hospitals except where the intensity of EM disturbances is high.

The MELODY Expert and MEV system are suitable to be used in home healthcare environment: eg. nursing homes, vehicles and emergency medical services.

## User training & warranty

Only users who have obtained signed permission from the company ADECOTECH may use the system. ADECOTECH is exempt from any warranty if the user has not been trained by the company.

## Maintenance

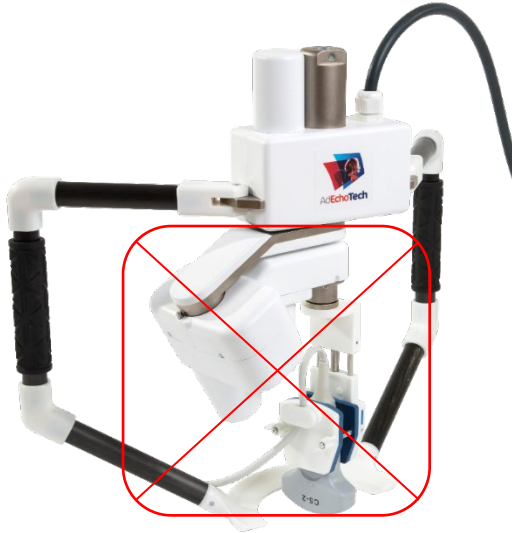
Maintenance of the MELODY system shall only be undertaken by the manufacturer or an organization approved by the manufacturer.

## System Manipulation

### *MELODY PATIENT*

- Always position the Patient control interface with the touchscreen facing upwards.
- Never hold or pick up the robotic probe holder by the umbilical cable or by the parts shown inside the red frame in the illustration below.
- During the examination and while the expert is taking control of the robotic probe holder, never try to take hold of the cable of the ultrasound imaging device probe.

- During the examination and while the expert is taking control of the robotic probe holder, never obstruct the robotic arm's field of action. In the event of a blockage in the robotic arm's field of action, the robot arm shifts and come back to its normal use after 5 seconds. This field of action is shown inside the red frame in the illustration below.

### **MELODY EXPERT**

- Always position the Expert Control interface with the touchscreen facing upwards.
- Never hold or pick up the Expert set by the cables.
- The fictitious probe is a delicate component of the system. Always handle it gently.

### **MEV**

- The fictitious probe is a delicate component of the system. Always handle it gently.

### **Recommandation for Use**

Before starting the examination, make sure that the following conditions are verified. If one these conditions is not followed, the examination **MUST NOT** be performed:

- The communication with the patient site through the videoconferencing system must be established,
- The real-time display of the ultrasound images must be effective,
- The expert must be able to take control of the probe holder robot.

At any moment of the examination, if the expert has any doubts about the normal behaviour of the MELODY system or thinks that the safety of the patient is not guaranteed, it is imperative to remove the system from the patient body and stop the examination by pushing the emergency shutoff.

## Precautions

Do not spray cleaning or disinfectant solutions directly onto the surfaces of the MELODY system; these solutions may penetrate the system and damage it.

Do not use solvents (paint thinners, benzene, etc.) or abrasive products since they may damage the external surfaces. During cleaning, make sure that the solution used does not penetrate inside the system.

The user is solely responsible for the degree of disinfection; this depends on the tissue with which the system comes into contact during the examination.

## Troubleshooting

For any technical assistance, our office hours are 9 am to 12 pm and 2 pm to 6 pm (UTC+1), Monday to Friday. You can contact us by telephone on **+33 (0)9 67 17 47 46** or via email to the following address: **info@adechotech.com**

In case of emergency, call **+33(0)6-72-05-40-18**.

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**ADECHOTECH**  
Bringing Ultrasound Everywhere

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